

DEPARTMENT OF ADMINISTRATION ASSISTIVE TECHNOLOGY STAR

Assistive Technology for Medication Management

Presented by:

Courtney Ness Fuchs, ATP

February 8th, 2023

AT for Med Management Considerations - Part 1

- Do they need a reminder, organizer/dispenser, or both?
- How many times per day do they take medication?
- If they need a dispenser, does it need to be locked?
- How big of a space is needed for the pills at each dose? How many pills per dose, and how large are the pills?
- Do they have medications that will not fit in a pill box such as creams or insulin?
- How is their dexterity? Will they be able to open a pill box to retrieve their medications?
- How is their vision?
- How is their hearing?





AT for Med Management Considerations - Part 2

- How is their attention span? If the person can retrieve their medication from the dispenser, do they still need prompts to take the pills? Are they likely to double dose?
- How is their memory? Are there special instructions they may not remember?
 i.e. take with food
- Does the person need help filling the dispenser? If so, how often is someone available to do this or to do a compliance check?
- Do they need a monitored solution that will alert someone immediately if they've missed a dose? Who needs to be alerted?



Reminder Watches

<u>TabTime Vibrating Reminder</u> <u>Watch</u>

• Up to 10 silent vibrating alarms/day



e-Pill Alarm Watches

• Multiple watch options available





Timer Caps for Pill Bottles

<u>TimerCap</u>

- Indicates how much time has passed since the bottle was last opened
- No programming



e-Pill TimeCap & Bottle

- Display indicates the last time/date the bottle was opened
- Can set alarms on the hour, up to 24 alarms/day







"Analog" Dose Trackers

<u>Take-n-Slide Medication</u> <u>Tracker</u>

- Peel and stick onto each pill bottle
- Slide the indicator for that day to indicate you've taken that day's dose


<u>Pill Popper Dose Tracker</u>

- Stretchy band can go around differen sizes of bottles
- Pop the indicator for that day to indicate you've taken that day's dose
- Additional indicators for 2nd and 3rd daily doses



PILL IT Scratch Off Stickers

- Peel and stick onto each pill bottle
- Scratch off the icon to indicate you've taken that day's dose

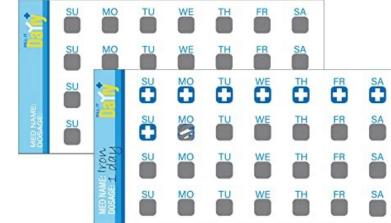



Basic Pill Organizers

EZ Dose Push Button AM/PM Pill Organizer



<u>e-Pill 5 Times a Day x 7 Day</u> <u>Large Pill Organizer</u>

e-Pill offers a wide variety of medication dispensers



SSIS

Tons of options available online. Search for the number of times per day. If dexterity is a concern, look at reviews to find an easier-to-open/arthritis-friendly option.

MedCenter System Monthly Pill Organizer

- Unlocked pill box organizer
- Up to 4 talking audio alarms per day
 - Alarms state the day, time, and dose to take.
- 31 numbered pill boxes each with four compartments: morning, noon, evening, and night.
 - Each compartment can hold up to 10 M&M-sized pills
- Pill boxes are coded red/green to indicate when the date has passed
- MedCenter offers different profile systems, i.e. extra-large, weekly, are available







Med-Q

- Unlocked pill box organizer
- Up to 2 alarms/day
- Alarm sounds and lights up the compartment to be taken at that time
- Alarm volume gradually increases
- Portable and battery-powered
 - Batteries should last about 1 years





DMINISTRATION

Dispenser + App Alerts

<u>Sagely Smart Extra Large Pill</u> <u>Organizer</u>

- 7 daily pill boxes with AM/PM slots
- Large compartments hold up to 10 large vitamins each
- Push through for easy filling
- Free companion app for <u>iOS</u> and <u>Android</u> to remind you when it is time to take your pills



ADMINISTRATION

Family/Caregiver Monitoring Option

MedReady 1700

- Locked, timed dispenser with audio alarm
- Flashing model available
- Holds 28 doses
- Up to 4 alarms/day
- Landline and cellular monitored models available
 - Receive notifications of missed doses via text message, email, or phone call
 - Monthly monitoring fee
 - Cellular model allows caregivers to change alarm times, trigger early doses, and change alarm times remotely.
 - Does not require Wi-Fi (uses landline phone or cellular towers)







Family/Caregiver Monitoring Option

Dose Flip

- Locked, timed medication dispenser with audio and flashing alert
- Portable size (5" diameter) & 3-4 day battery life
- Receive notifications of missed doses via text message, email, or phone call
- Online dashboard shows all device history, incl. tampering. A monthly report can be sent.
- Adaptive equipment available for free
 - Stand to make flashing light easier to see
 - Adaptive flipper for those who struggle with picking up the device
 - Extra trays for pre-filling
- \$60/month monitoring fee
- Does not require Wi-Fi (connected via cell towers)
- Offers free trials info@dosehealth.com



ADMINISTRATION ASSISTIVE TECHNOLOGY - STAF

ASSIS⁻

App Alerts + Family/Caregiver Monitoring Option

Medisafe App

- Enter all of your medications to receive a reminder for each
- Reminders can be customized to use fun voices such as Darth Vadar and Elsa
- Alerts a family member or caregiver if a dose is missed
- Alerts to drug interactions
- Refill reminders
- Free app for <u>iOS</u> and <u>Android</u> with paid option available for many additional features

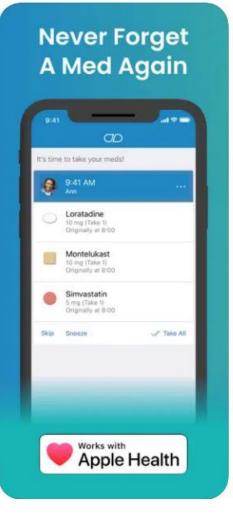




Image credit: <u>howtodrawforkids.com</u>




DEPARTMENT OF ADMINISTRATION ASSISTIVE TECHNOLOGY - STAF

App Alerts

iOS 16 + watchOS 9 Health App Medication Feature

- Add each medication and supplement you take, including form and strength.
 Select the pill shape and color.
 - Add meds by searching or by using the phone's camera
- Log medications from the Health app on your iPhone or from the Medications app on your Apple Watch
- Receive reminders that it is time to take meds via notifications on your iPhone or Apple Watch
- See potential drug interactions
- Export a copy of your current med list
- Review up to 28 days' worth of medication compliance history

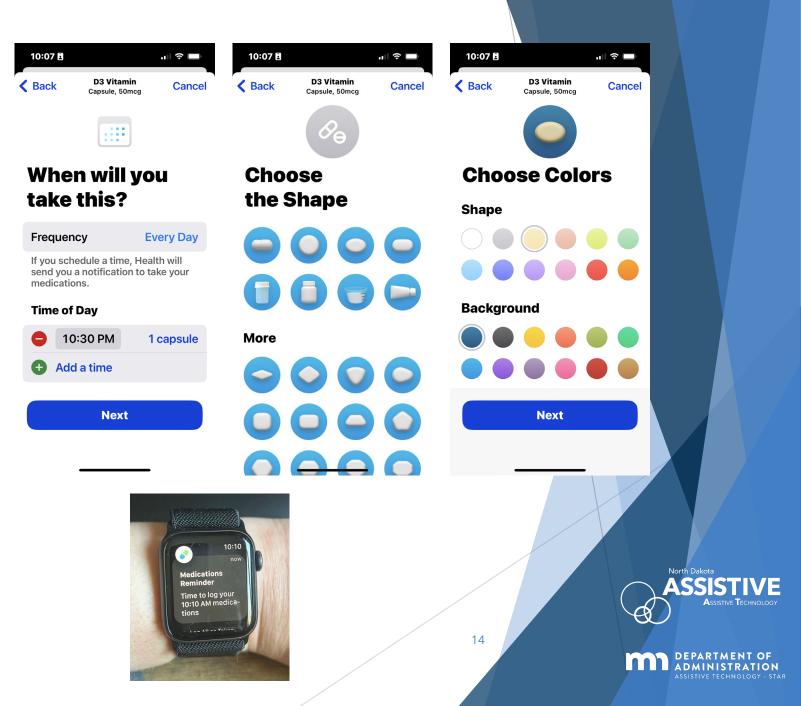

Use Your Smart Assistant!

Alexa, Google Assistant, Siri, Bixby, oh my!

Image credit: Warner Brothers

ASSISTIVE TECHNOLOGY

15

DEPARTMENT OF ADMINISTRATION ASSISTIVE TECHNOLOGY - STAR

Alexa Reminders

- Will announce and display the reminder from the Alexa device(s) of your choice
- Can announce once, twice, thrice, or persistently for up to an hour
- Can send notifications and text messages to a smartphone
- Can be set to announce at a time OR a location
- Custom repeat frequency daily, weekly, monthly, etc.
- Can be set to repeat up to 24 times/day
- If there are multiple Alexa household members, you can select who the reminder is for

11:48 🖪				. II 🗢 		
Cancel	ADD REMINDER			Save		
Remind me to Take your evening meds						
At a time	2		🔿 At a	a location		
Announces f		Kitchen Echo Show				
Start date			We	ed, Jan 25		
Repeat				Daily		
Time				10:30 PM		
	~ 7	26 27				
	8	28				
	9 10	29 30	AM PM			
	11	31	r" IVI			
	12	32				
	1	33				
Reminder fo				0		
	r	C	ሰ	Cour <u>tne</u> v		
Home Communic	ate	Play	Device	More		



Alexa Routines

- Completely customizable!
- Set up a medication reminder routine that will be triggered at a certain time(s) each day
- Alexa can announce a custom message, i.e. "Take your Vitamin D" or "Take ½ of a tab of your Metformin with a full glass of water"
- Add additional supports such as a song, bell for attention, or a follow-up reminder 3 minutes later

10:54 🖪		
<	EDIT ROUTIN	E :
Enabled		
NAME		
Vitamin D		Change
WHEN IT'S		
7:28 AM Every Day		
ALEXA WILI	-	
Add actior	ı	•
Say "T	ime to take y	vour Vi =
Play "I	'm Walking c	on suns =
FROM		
Kitchen Ec	ho Show	0
Home	Play	Devices More



Links to Other Smart Assistants

- ► <u>Google Assistant</u>
 - ► <u>Google Assistant Reminders</u>
 - ► <u>Google Assistant Routines</u>
- ► Siri/iOS
 - ► <u>iOS Reminders</u>
 - ► iOS Automation in Shortcuts
- Bixby
 - ► <u>Bixby Reminders</u>



Google Assistant





ScriptAbility Accessible Prescriptions Labels for the Visually Impaired

ScripTalk Talking Labels



ScriptView

John J. Smith Amoxicillin 500mg Tab Take one tablet three times



DEPARTMENT OF ADMINISTRATIO

Other AT for Medication Help

"This image depicts AutoDrop and AutoSqueeze being used together



Eye drop applicators

Pill pack punchers

Long-handled lotion/ointment applicators

DMINISTRATION

Try Before You Buy!

Free demonstrations and short-term loans of various medication dispensers and reminders are available through your state's AT Program!

Minnesota Short-Term Loans



21

North Dakota Short-Term Loans





ADMINISTRATION

Potential Funding Avenues through ND Assistive

Contact ND Assistive to discuss your client's unique situation. We will guide you through possible funding sources.



ND Residents Senior Safety Program

- ► To qualify you must:
 - Be a North Dakota resident;
 - Be age 60 or over; and
 - Not live in a skilled nursing facility.
- Receive free assistive safety devices to help increase safety and independence in your home
 - Examples: anti-elopement devices, alerting devices for hearing loss, grab bars, bathtub rails, handheld shower heads, shower chairs, raised toilet seats, toilet safety frames, medication reminders and dispensers, personal hearing amplifiers, adaptive silverware
- Request Form is available on <u>ND Assistive's website</u>
- This project is supported by funding from the U.S. Department of Health & Human Services, Administration for Community Living, Administration on Aging, and granted ²³ through the North Dakota Department of Human Services, Aging Services Division.






ND & Moorhead, MN Residents **Possibilities Grant Program**

- Open to ND or Moorhead, MN residents with disabilities who meet income guidelines.
- Purchases assistive technology devices and services - can provide partial payment
- Grants do not exceed \$2,000
- Applications are reviewed by a committee as they are received.
- Applications are available on <u>ND Assistive's</u> <u>website</u>



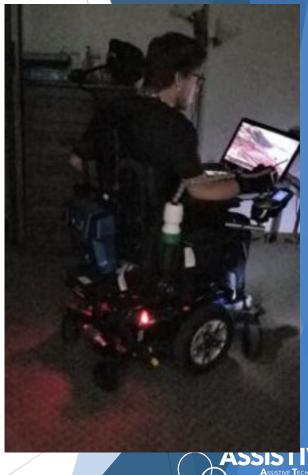

Contact Us - North Dakota

Courtney Ness Fuchs, ATP

Assistive Technology Training Specialist North Dakota Assistive cness@ndassistive.org



North Dakota Assistive

www.ndassistive.org 800-895-4728 https://www.facebook.com/NDAssistive/

ND AT4ALL https://nd.at4all.com/

Bismarck Office 4501 Coleman St, Suite 107 Bismarck, ND 58503

Fargo Office 3240 15th St S, Suite B (south door) Fargo, ND 58104



Contact Us - Minnesota

Jamis Wehrenberg Assistive Technology Specialist Minnesota STAR Jamis.wehrenberg@state.mn.us





Minnesota STAR

https://mn.gov/admin/star/ 888-234-1267 Relay: 800-624-3529 https://www.facebook.com/MNSTARProgram

MN AT4ALL https://mn.at4all.com/

Main Contact

STAR Program 358 Centennial Office Building 658 Cedar Street Saint Paul, MN 55155

